

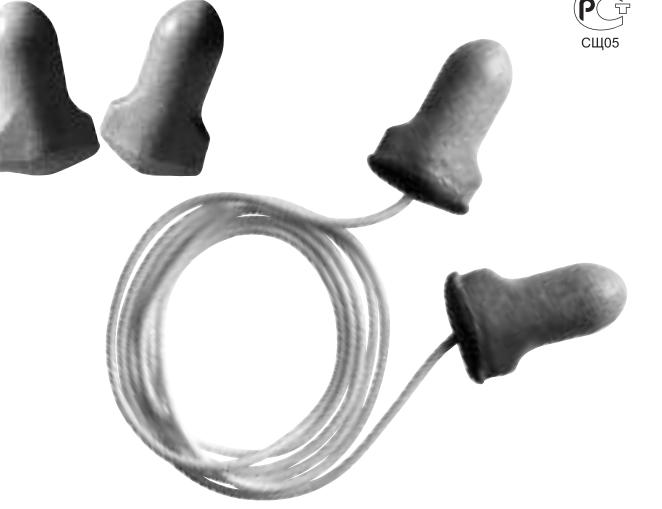
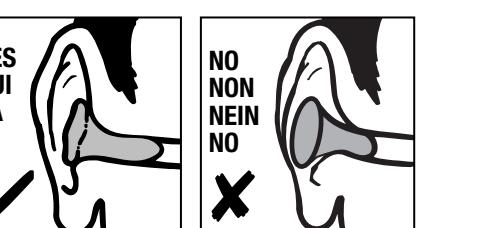
MAX LITE®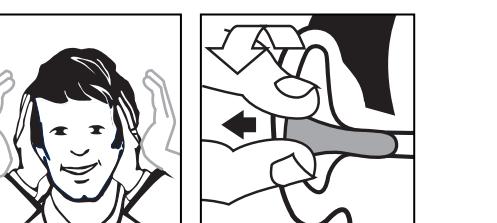

CE
EN 352
P.S.
CS105

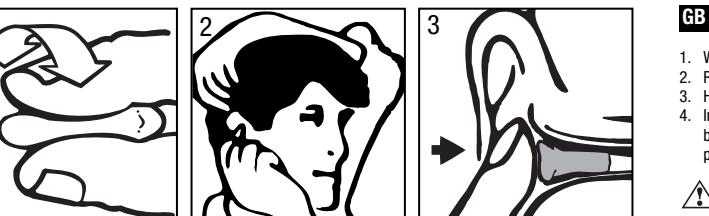HOWARD LEIGHT

by Honeywell

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GB US AU NZ

FITTING INSTRUCTIONS

- With clean hands, ROLL entire earplug into narrowest possible crease-free cylinder.
 - Reach over head with free hand, pull ear up and out and INSERT earplug well inside ear canal.
 - HOLD for 40 seconds, until earplug fully expands in ear canal.
 - In a noisy environment, with earplugs inserted, cup hands over ears and release. Earplugs should be blocking enough noise so that covering them with hands does not result in a significant difference. If proper fit is not obtained, move to a quiet location and repeat fitting instructions.
- WARNING!** Remove earplug slowly with twisting motion to gradually break the seal. Rapid removal may damage eardrum.
- WARNING!**
- (a) Please ensure that the earplugs are fitted, adjusted and maintained in accordance with instructions. Improper fitting will reduce effectiveness and could result in hearing loss or injury.
 - (b) Earplugs should be worn at all times in noisy surroundings.
 - (c) This is a disposable earplug. It is recommended that this earplug not be reused. Earplugs should be cleaned by removing all earwax and contaminants prior to re-insertion. Soiled plugs may be wiped clean with a moist cloth. Do not use solutions, disinfectants or chemicals.
 - (d) This product may be adversely affected by certain chemical substances. Further information should be sought from the manufacturer.
- If the recommendations given in paragraphs (a) to (d) above are not adhered to, the protection afforded by the hearing protector will be severely impaired.
- WARNING!** Keep away from infants and small children. These earplugs are non-toxic but may interfere with breathing if caught in windpipe, which could lead to serious injury or death. The cord attached to these earplugs is not a toy. It may interfere with breathing if wrapped around the neck, which could cause serious injury or death.
- WARNING!** These earplugs should not be used where there is a risk that the connecting cord could be caught up during use.

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©2014 Honeywell Safety Products USA, Inc.